

Safety Instructions

IMPORTANT: Follow these basic safety precautions to reduce the risk of fire, electrical shock and/or injury when using this electrical appliance.

- This product is NOT a blanket. It is intended for use ONLY as a massage table warmer.
- DO NOT use with pets.






DIRECTIONS FOR USE:

1. Unfold completely.
2. Check frequently to make sure the table warmer is NOT bunched or folded as this can cause a build up of excess heat and result in injury or fire.
3. If at any time the table warmer is not functioning properly, discontinue use immediately and consult the manufacturer or distributor.
4. Conduct routine damage inspections of the table warmer and its cord, plug and power supply. If damage is detected, discontinue use immediately and consult the manufacturer or distributor.
5. DO NOT allow the cord of the table warmer to become pinched or damaged by leg extensions or lift mechanisms, when using with an adjustable height massage table.
6. DO NOT allow the table warmer to hang over the edge of the massage table.
7. DO NOT leave disabled persons, infants or small children unattended when the table warmer is in use.
8. Long treatments conducted with the table warmer on its high setting can result in skin irritation or burns.
9. DO NOT allow the control unit to come in contact with liquids (solutions, drinks, liquid cleaners, wet environments, etc.)
10. DO NOT puncture the table warmer with sharp objects.
11. Use table warmer only on an alternating current supply circuit of correct voltage.
12. Keep the flexible power supply and switch off the top of the table.
13. When not in use, unplug the unit at the power outlet.
14. Follow cleaning instruction on the product label.



Operation & Controls

1. Turn table warmer on.
 - Insert table warmer plug into the wall outlet.
 - Press "ON/OFF" button and hold for 2 (two) seconds. Heating indicator will appear on upper left side of screen.
2. Set controller temperature
 - A thermometer mark will be present at the top of screen.
 - Adjust "+" or "-" to chose your prerred heating temperature.
 - Available temperature range is 86°F - 130°F.
Note: some controllers can go up to 135°F
3. Set heating time.
 - Press the "SELECT" button once and the clock icon will appear at the top of the screen.
 - Adjust "+" or "-" to choose the preferred amount of heating time. Available time range is 1 minute (E-01) to 99 minutes (E-99).
4. Locking your settings
 - Press and hold the "LOCK" button for 2 (two) seconds. This will lock the set heating temperature and time. Lock indicator will appear on screen.
 - Press and hold the lock button to unlock.
5. Obtaining current status
 - Press the "STATUS" button once to obtain the actual, current temperature and current heating time (if previously set).
6. Turn table warmer off.
 - Press and hold "ON/OFF" button for 2 (two) seconds to turn controller off.

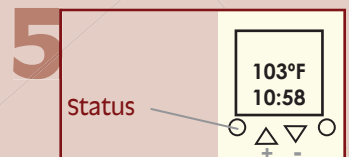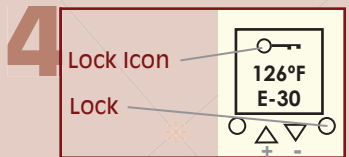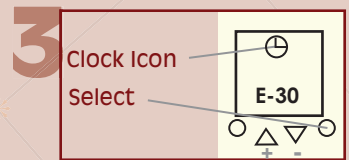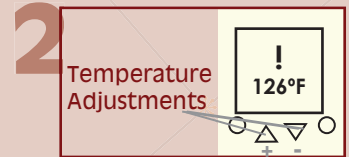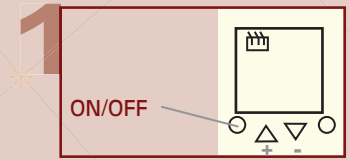

Cleaning & Storage

- When storing, fold the table warmer loosely, avoiding sharp bends and creases in the heating elements.
- DO NOT roll the table warmer
- DO NOT store objects on top of the table warmer
- DO NOT dry clean. Chemicals can damage the insulation and the cover of the heating unit.
- DO NOT iron.
- DO NOT machine wash or dry.

To Clean:

1. Before cleaning, make sure the hand control is set to the OFF position.
2. Disconnect the electrical cord from the power outlet.

3. Spot clean by wiping the soiled area with luke warm water and a neutral detergent or mild soapy solution.
4. Wipe again with clean water.
5. Blot away excess water by compressing the wet area between two dry towels.
6. DO NOT allow water to drip on the control unit.

Drying:

1. Drape the table warmer over a clothes line and drip dry.
2. Do Not use clothes pins or clamps.
3. Do Not dry with a hair dryer or heater.
4. When hanging make sure to position the hand control so that no water will drip onto the control unit.